test.zmdx.at

SARS-CoV-2 PCR TEST

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ENGLISCH

INSTRUCTIONS FOR USE: PCR TEST KIT with mouth/throat rinse for self-testing.

These instructions for use contain general information and instructions for use for the independent collection of a saliva sample for a molecular biological test procedure. The sample collection procedure and the verification process to be carried out prior to sample collection are also explained. To perform the PCR test, you need a valid e-mail address, an internet-enabled computer with webcam or a smartphone, and an official photo ID. Please read the instructions carefully.

INTENDED USE

This PCR Test Kit is used for the qualitative detection of the novel SARS-CoV-2 virus in saliva samples and supports the diagnosis of infection with the SARS-CoV-2 virus. The PCR Test Kit includes the set for self-testing for sample collection and the molecular biological laboratory service. The sample collection is carried out by the user (self-testing) according to these instructions for use. The molecular biological analysis of the sample using the RT-PCR method is carried out by the certified laboratory Zentrum für molekulare Diagnostik GmbH or by a recommended partner laboratory. After molecular biological analysis, the user receives a notification of the result via SMS. A digital Covid-19 certificate is available for retrieval on the web app: <https://test.zmdx.at>.

WEBAPP

Open the web app: test.zmdx.at and follow the instructions. These are always up to date and may differ from the printed instructions for use. The entire PCR test kit as well as the sample collection procedure and verification are explained in detail in the web app. Please note that results cannot be transmitted without registration in the web app. An internet-enabled computer with webcam (PC/laptop/tablet) or a smartphone and a valid e-mail address are required to use the web app.

CONTENT OF THE PCR TEST KIT

See Figure 1.1.

INSTRUCTIONS FOR USE

1. ACTIVATE PRODUCT

1.1. Verify the PCR test kit with sample number and e-mail address

After initial registration on the portal, you will be asked to verify yourself by e-mail. You will receive a link for verification. Please open this and also check your spam folder. You have the option to skip the verification e-mail at this stage; however, please note that the presence of a valid e-mail address is mandatory in order to receive the test result.

1.2. Enter your personal data

Please make sure that you enter your personal data completely and correctly, as this is a prerequisite for the evaluation and delivery of the results.

1.3. Proof of identity is necessary to obtain a certificate valid in the EU (green pass). For proof of identity, it is necessary to allow access to your camera and have proof of identity (passport, identity card or e-card) ready. Scan your ID with the camera of your smartphone or webcam. The test will take place in front of a running camera. You also have the option of skipping the proof of identity, but please note that in this case a digital COVID certificate (green pass) valid throughout the EU cannot be issued. Only persons who have completed the proof of identity will receive a digital Covid-19 certificate with their medical report.

2. CARRY OUT SAMPLE COLLECTION

2.1. Collecting a sample

On the web app you will be instructed how to prepare and perform the sample collection (rinsing/gurgling process) in the pharyngeal cavity. Figure 1.1. explains the contents of the PCR test kit. Do not eat or drink anything for 30 minutes before taking the sample. Open the saline solution (1), put it in your mouth and rinse/gargle thoroughly for at least 1 minute. Take the transfer tube (2) and spit the rinsed saline solution into the opened test tube (3). Then close the test tube with the cap.

2.2. Packing the test kit

Place the sealed sample tube into the protective bag (4). Make sure that the cellulose wadding (5) is in the protective bag. Now close the protective bag well and put the closed

protective bag back into the PCR test packaging (6).

IMPORTANT: Keep the instructions for use (7) with your sample number in a safe place and never put it in the PCR test kit.

2.3. Return the test set

Take the PCR test package to a possible drop-off point immediately after sample collection. Consult the web app.

2.4. Disposal

Dispose of the transfer tube with the paper waste; dispose of the empty saline ampoule with the packaging waste.

TEST RESULT

After molecular biological analysis, the user will receive a notification of the result via SMS. The link mentioned in the SMS will take you to the digital Covid-19 certificate. The laboratory is legally obliged to register all results in the national epidemiological reporting system (EMS). In case of a positive result, the competent authority will contact you directly.

Possible test results are:

DETECTED

(infected with SARS-CoV-2/positive)

NOT DETECTED (NOT infected/negative)

INVALID (sample could not be analysed)

GENERAL INFORMATION

Storage: Store between 4°C - 25°C.

Content: 2.5ml/5ml 0.9% NaCl solution.

Expiry: see packaging.

This is a product for in vitro use. You can find the current version of the instructions for use at <https://test.zmdx.at/downloads>. This is a single use product; reuse is excluded. Further information on the analysis procedure, specificity, measuring range and reference values can be found at <https://test.zmdx.at>. If you have questions of a medical nature, consult your doctor.

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Procomcure Biotech GmbH